Artelac® Ectoin® Drops

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www.bausch.com

Please carefully read these instructions for use – they contain important information on how to use this product. If you have any questions, please ask your pharmacist, doctor or contact us. EN

BAUSCH + LOMB

Artelac® Ectoin® Drops

Composition: 0.24% hyaluronic acid (as sodium hyaluronate), 2% Ectoin®, sodium chloride, sodium hydroxide (for pH adjustment) and water for injection

Intended use: Supportive treatment and prevention of symptoms of allergic conjunctivitis, relief of inflammatory-irritated eyes and soothing eye lubrication

What is Artelac Ectoin® Drops?

Artelac Ectoin® Drops is a Medical Device which contains Ectoin®, a natural cell-protective molecule with inflammation-reducing and membrane stabilizing properties and hyaluronic acid, a natural molecule which comforts the eye by its high water-binding and stabilizing characteristics. It is free of buffers.

The advanced formulation possesses a very high tolerability, as it is free of preservatives.

Indication for use: Artelac Ectoin® Drops is an isotonic eye drop solution which can be used to treat and prevent symptoms of seasonal and perennial allergic conjunctivitis caused by pollen, dust mites, animal hair or mold, etc. Symptoms, such as itchy, irritated, red or watery eyes will be reduced effectively; their soothing lubrication will result in relief, even of inflammatory-irritated eyes. Artelac Ectoin® Drops reduces the inflammation of the conjunctiva due to chemical-physical stimuli via non-pharmaceutical action.

When you should use Artelac Ectoin® Drops?

Artelac Ectoin® Drops stabilizes the natural tear film in two ways: hyaluronic acid has a positive impact on the aqueous phase and Ectoin® strengthens the lipid phase. This dual mode of action protects the eye against the harmful influences of allergens and other external factors and supports the regeneration process of the irritated and sensitive conjunctiva. Artelac Ectoin® Drops is free of preservatives and suitable for wearers of contact lenses and those with sensitive eyes.


How often and how long should you use Artelac Ectoin® Drops?

Instill 1-2 drop of Artelac Ectoin® Drops into the eye as needed. You can use it every day and as often as you want. There is no limit to the duration of use.


How to use Artelac Ectoin® Drops?

The patented pump mechanism of Artelac Ectoin® Drops allows the drop solution to be preservative free, but it may not look like other drop bottles you have used in the past.


General note: Wash your hands before each use. Avoid contact of the tip of the bottle with your fingers or your eyes to prevent any possible contamination of the solution or eye injuries.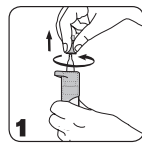


1. Before each use, remove the protective cap from the tip of the bottle with a rotating movement.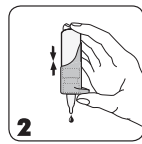

2. **Attention! Before first use and if the bottle has not been used for a while:** Point the bottle with the tip downwards and press the pump mechanism repeatedly until a drop is released.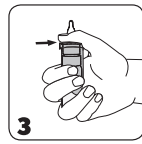

3. Hold the bottle as indicated: Place your thumb on the ergonomic thumb support and enclose the lower part with your fingers. Two fingers should be placed on the bottom of the bottle.

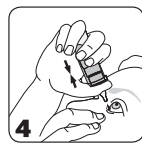4. Point the bottle with the tip downwards. Lean your head back slightly. Gently pull your lower eyelid down with a finger of your free hand. Hold the bottle in a vertical position over your eye as shown. Gently press the pump mechanism to apply one drop into the eye.

Close your eye after instillation and blink to optimally spread the drop across the surface of your eye.

5. Immediately after use, shake-off any possible residual drops from the bottle and replace the protective cap on the tip of the bottle.

Due to the bottle design, a small amount of solution remains in the bottle. However, the specified filling volume will be completely dispensed from the bottle.

When you should not use Artelac Ectoin® Drops

Do not use this product if you are hypersensitive to any of the ingredients.

Important information about using Artelac Ectoin® Drops

- Local hypersensitivity reactions have been very rarely reported. Treatment should be discontinued in such cases.
- If you experience persistent discomfort or irritation, stop using the product and consult your doctor.
- Please note that prior to the first usage or after a period of non-usage, an initial pumping of the bottle pump mechanism is necessary. By pumping the solution, the bottle should be kept with the dropper tip down.
- This product may cause temporary blurred vision after instillation which can have a negative impact on driving vehicles or operating machines.
- Do not touch the eye surface with the tip of the bottle to prevent any possible injury of the eye surface or contamination of the solution.
- Do not touch the tip of the bottle with your fingers to prevent any possible contamination of the solution.
- Once opened, do not share the bottle with anyone else.
- Careless handling of the product contrary to the instructions for use could result in slipping of the product from the hand and may cause eye injury.
- Shake-off any possible residual drops from the bottle after application.
- Wait approximately 15 minutes before using another eye product.
- Discard the medical device after 6 months once opened.
- Do not use after the "use-by-date".
- Do not use if the package or bottle is damaged.
- Artelac Ectoin® Drops contains small plastic parts (protection cap) that could be swallowed by small children. Danger of suffocation. Therefore, Artelac Ectoin® Drops should be kept out of reach of children.
- Artelac Ectoin® Drops is suitable for use with soft and rigid contact lenses.
- This device can be used by lay persons not specifically trained.
- Based on current knowledge, the product can be used for pregnant, breastfeeding women and children.
- If your child is experiencing dry eyes, consult a doctor before using an eye lubricant.

How to store Artelac Ectoin® Drops: Store at a temperature between 2 - 25°C

Disposal: The product must be disposed of in accordance with local regulations.

Every serious incident that occurs in relation to the product should be reported to the manufacturer and the competent authority.

Presentation: Artelac Ectoin® Drops is available in packs of 10 ml eye drops.

Artelac Ectoin® Drops is available in convenient, preservative free single dose units (SDU) as well.

Explanation of Symbols

 **Manufacturer**
Dr. Gerhard Mann
chem.-pharm.
Fabrik GmbH
Brunsbütteler
Damm 165/173
13581 Berlin
Germany

 Sterilized using aseptic processing techniques and packaged in a single sterile barrier system.

 **UDI** Unique Device Identification
GTIN (01) device identification
EXP (17) use-by-date (format: yymmdd)
LOT (10) lot number

 **Temperature limit**
2°C - 25°C

 **MD** Medical Device

 Do not use if package or bottle is damaged

 **Caution** (consult instructions for use for important cautionary information)

 Can be used for 6 months after first opening

 **Date of manufacture**

 Consult instructions for use

 **CE** 0483
European Medical Device Conformity Assessment
CE mark with notified body number

 Use-by date

 **LOT** Batch code

Date of issue: **Artelac Ectoin® Drops, "I228EU-04"; national revision date (2023-09)**

Ectoin® is a trademark of Bitop AG. All other ®/™ are trademarks of Bausch & Lomb Incorporated or its affiliates

© 2023 Bausch & Lomb Incorporated or its affiliates

P1GR06 43156PB228/3-GR